

Nidek RS-330 Retina Scan Duo

Setup


Make sure both computers (Navis and Optimize) are on the same network.

Optimize

Create a shared folder on the Optimize PC for folder capture (e.g. shared as Navis).


Add a Folder Capture device pointing to the location of the shared folder.

Select the option to clear the folder on start.

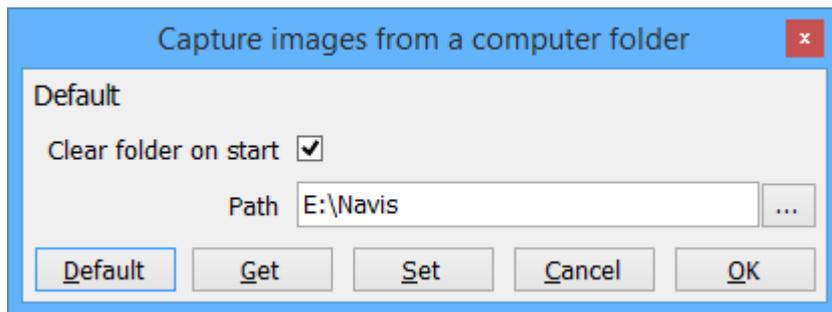


Navis


Go to Setup/Settings

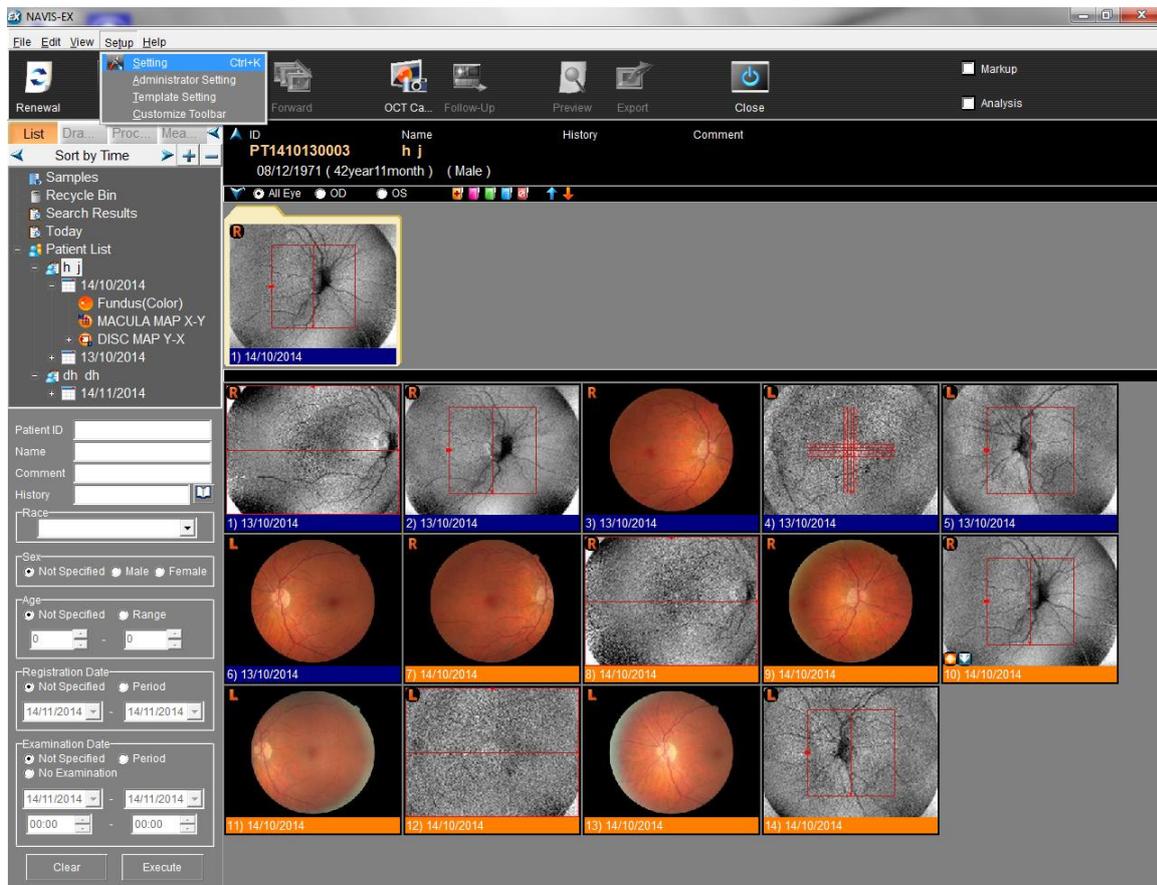


Select the Export tab.

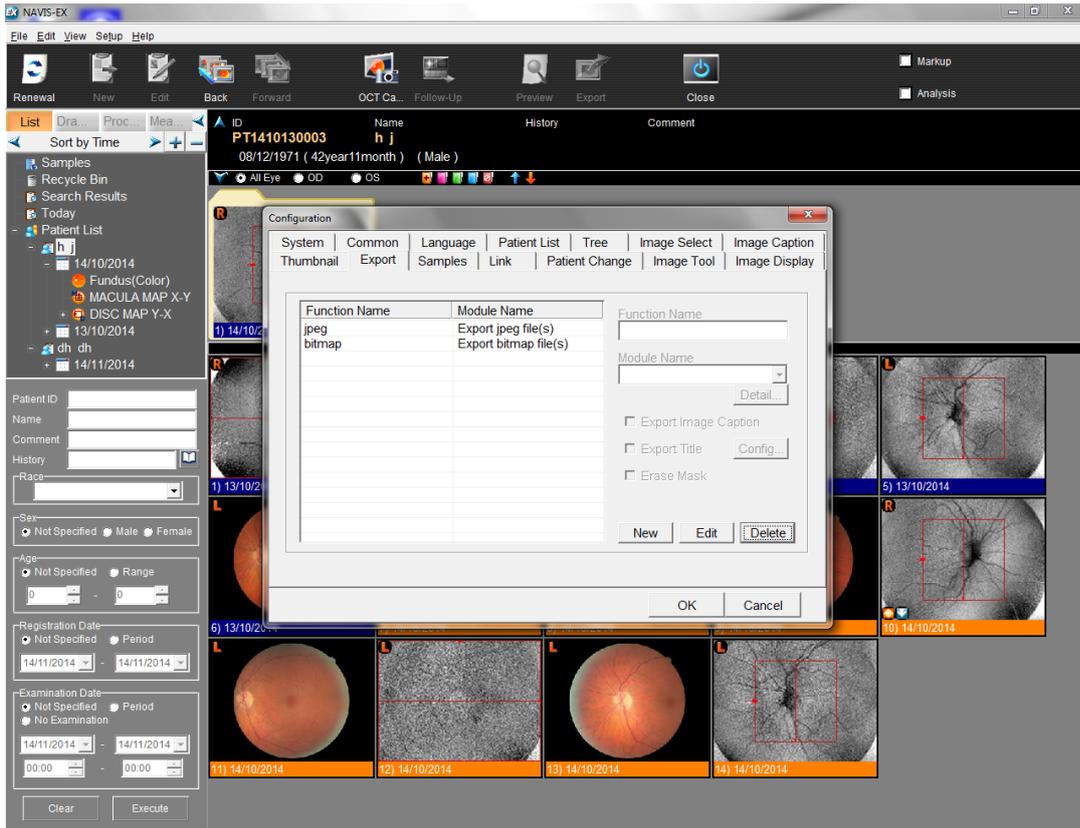

Click the New button and fill in the “Function Name” as Optimize and “Module Name” as Export JPEG file(s).

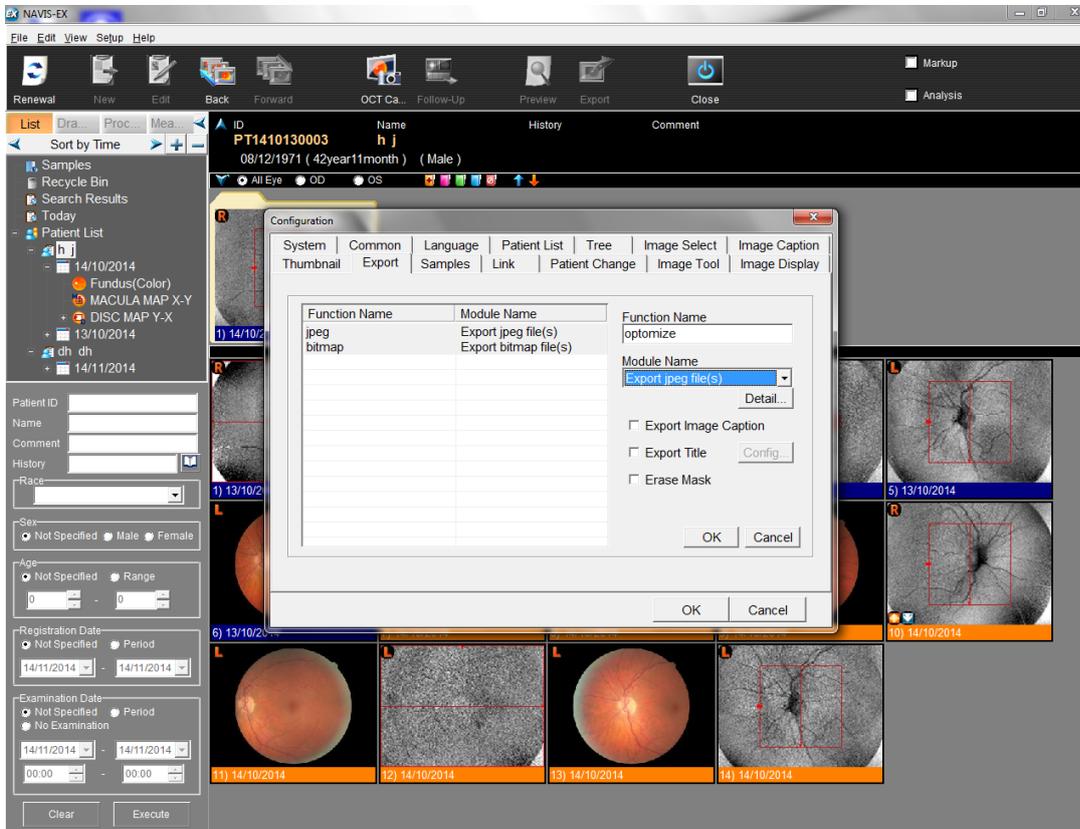

Click the detail button and enter the shared folder location into the Folder box and set the quality to 100.

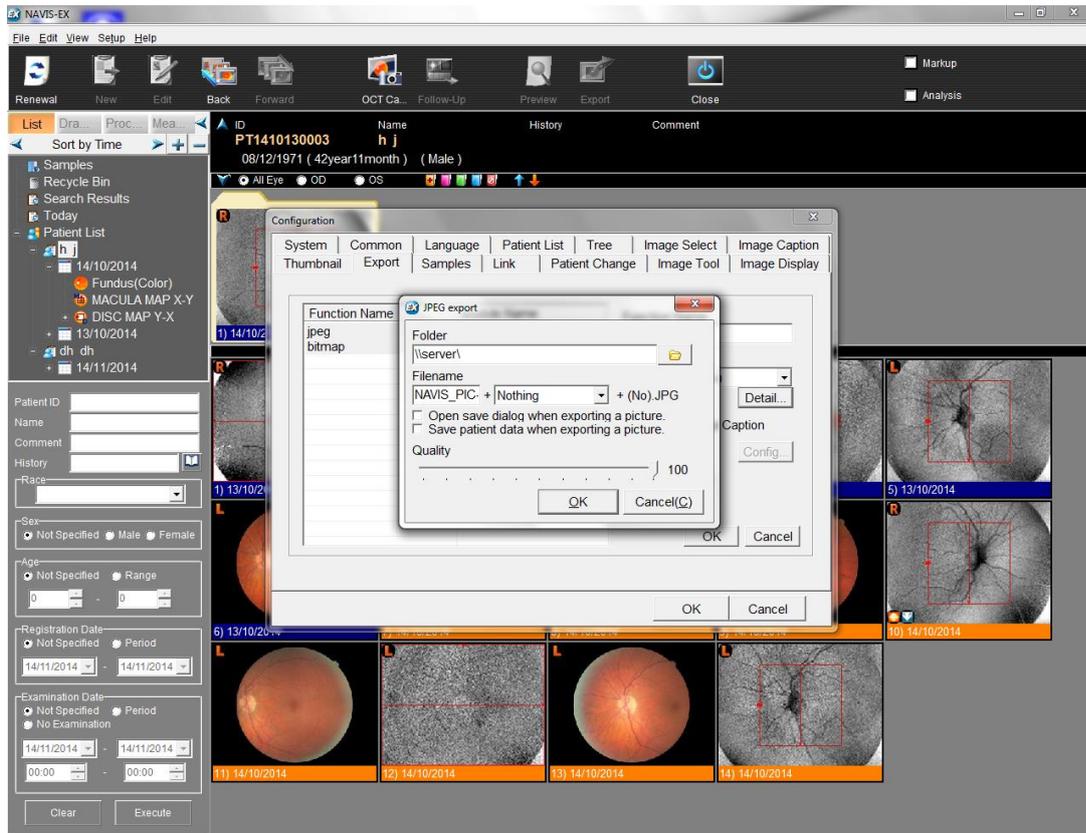


Operation

Optimize

Start a new photography session for the correct patient.

Navis

Highlight all the pictures you want to export. Click Export and select the Optimize option. These images will be picked up automatically by Optimize and will be deleted from the Navis shared export folder.

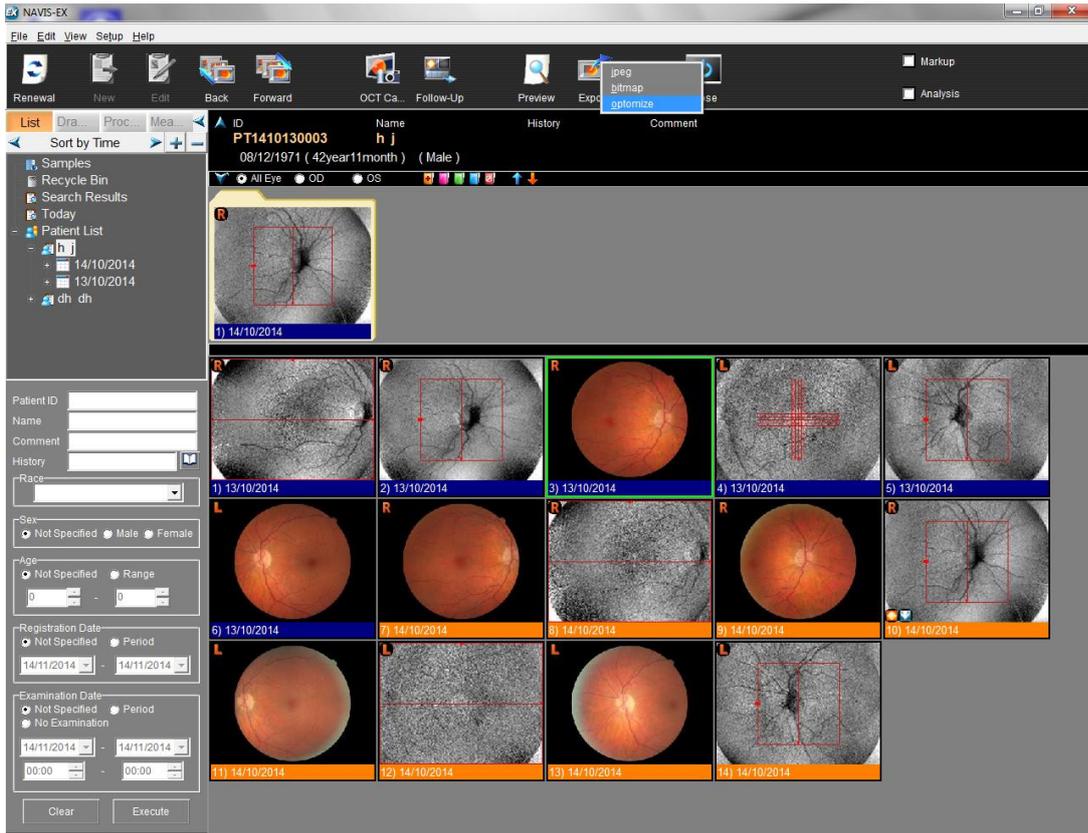

When the images have been exported you will see the following:

