

RT-6100 Quick Reference Guide

<Distance/Near>

Toggles between the distance vision test **FAR** and near vision test **NEAR**.

<Side menu>

Displays the side menu.

<Sub windows 1&2 (right/left) >

- Displays the previously set data.
- Displays the data set before the previous setting.

<Program message area>

Displays the refraction procedure.

<Current data area>

Displays the data of the lens currently in place.

<Chart display>

Displays the currently presented chart.

<Auxiliary function selection> Availability depends on charts.

- Changes the contrast.
- Displays charts in inverted black and white.
- Presents charts in night use brightness.
- Turns the glare lamp on or off.
- Changes the horizontal line letters randomly.
- = +

<Numeric entry>

, Changes the selected value.

<Data entry>

Displays the import data select screen for AR or LM data.

<Program advance>

- Starts a refraction program.
- Advances to the next test while a program is running.
- Returns to the previous test.

<Shift function>

Pressing a button or turning the dial while holding changes functions or increments.

<Window Open/Close>

, Covers or uncovers the measuring window (right/left).

<Dial>

Changes the selected value.

<Dial button>

Changes the mode to SPH, CYL, AXIS, or VA.

<Measurement mode>

Selects the mode for data entry and measurement.

<Day/Night>

Toggles between the day vision test and night vision test .

<Auxiliary lens selection>

Displays the auxiliary lens select window (right/left).

<Chart selection>

Select the chart to be presented.

<Program selection>

Displays the program select window.

<Favorites registration>

Displays icons for charts added as favorites.

<Help button>

Displays the explanation for the presented chart.

<Patient's eye selection>

Selects the right eye, binocular eyes, or left eye.

<Output>


- Exports or prints data.
- Clears data.

<Cross cylinder switching>

, Places the cross cylinder lens and enters AXIS or CYL mode.
This button is used to flip the cross cylinder lens in AXIS or CYL mode.

<VA chart masking>


- , Isolates a vertical column. Pressing the button again isolates its left or right column.
- , Isolates a horizontal row. Pressing the button again isolates its upper or lower row.
- Isolates a horizontal row.
- Isolates a single letter.

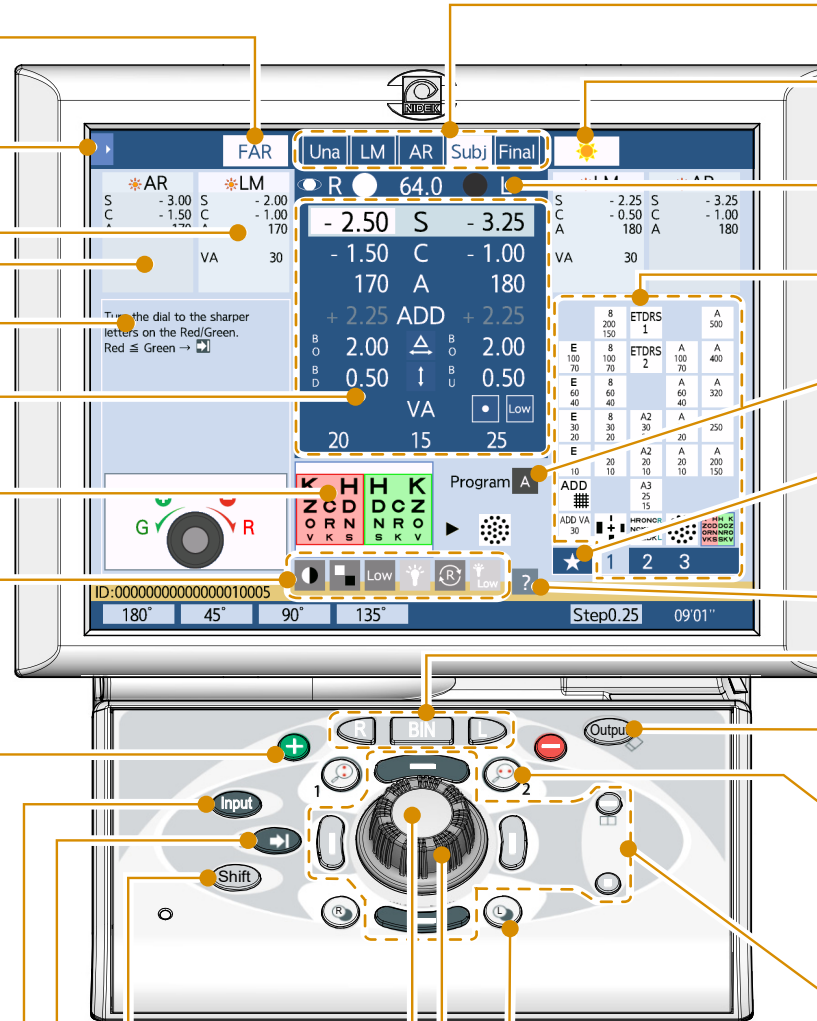

Side Menu

◆ The side menu displayed by pressing  on the measurement screen enables operations other than measurement.

<Close>

 Closes the side menu.

<ID>

Used to enter the patient ID.

<Age>

Used to enter the patient's age.

<Data list>

Displays the measurement data list.

<Range of Clear Vision>

Displays the clear vision range check screen.

<[+], [-]>

Selects a cylinder reading sign from plus and minus.

<Lamp off>

Turns off the chart lamp, touch screen backlight, and near point lamp.
To turn them on, press any button.

<Ch. Lamp>


Turns the chart lamp on or off.

<About Eye>

Displays the Refraction diagram list screen.

<Image>

Displays images saved on the SD card.

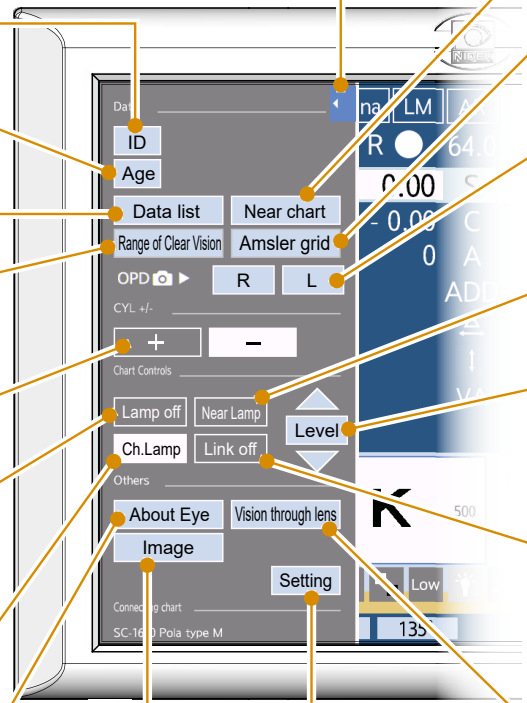

<Near chart>

Displays the near chart screen.

<Amsler grid>

Displays the Amsler grid screen.

<[R], [L]>

Displays OPD images of the right or left eye (only when data is imported).

<Near Lamp>

Turns on or off the near point lamp.

<Level>

Used to align the chart height to the patient's eye level (only when the SSC-370 is connected).

<Link off>

Disables auxiliary lenses or mode (SPH, CYL, AXIS) from being automatically switched when the chart is changed.


<Vision through lens>


Displays the vision image screen.

<Setting>

Displays the menu screen.


Favorites Chart Registration

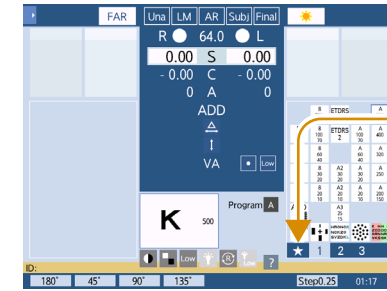


1 Press .
Displays the Favorites area (blue frame).



2 Press .
Displays the Edit area (green frame).


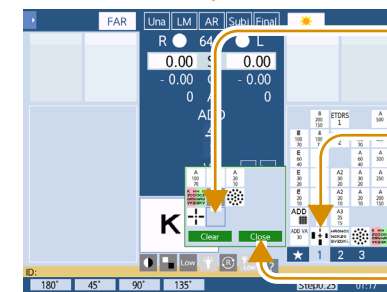


3-1 Press the position where a chart is to be placed. A cursor is displayed.

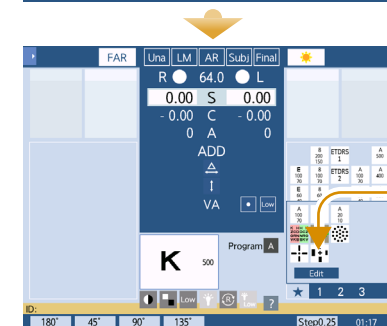

3-2 Select the icon of the desired chart. The chart is added at the cursor position.

3-3 Press .

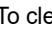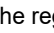To clear the registration, press , select the chart to be cleared, then press .

4 The chart is added to favorites.